User's Manual

Please read carefully

Manuel D'Instructions

Veillez lire attentivement

Manta 132 Rescue Aid

U.S. PATENT 5,214,813
OTHER PATENTS PENDING

BREVET AUX E-U 5,214,813
AUTRES BREVETS EN INSTANCE

FEATURES and BENEFITS:

- **Specially designed handles**
 - handles interlock by placing handles together and pushing one flap through the other.
- **Pockets at both ends**
 - holds victim's feet or head
 - holds back board
- **Eases transfer of patient**
 - victims can be transferred directly onto wheeled cot into the ambulance and onto hospital without removal from the stretcher
- **Confined space rescue**
 - adapts to bathtubs, bedrooms, corners, narrow hall ways, stair landings, etc.
- **Material**
 - fibre reinforced vinyl
 - barrier to body fluids
 - no stitching
 - Fire retardant
 - conforms to NFPA 701/ULC S-109
 - maximum load limit 800 pounds (363 kg), utilizing all 8 handles

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Manufactured in Canada by / Fabriqué au Canada par:
J.T.W. Rescue-Stretcher Inc.,
R.R. 1, Lakefield, Ontario K0L 2H0

Distributed by / Distribué par:
Ferno-Washington, Inc.
70 Weil Way, Wilmington, Ohio 45177

MANUAL

The Res-Q-Mate Manta is to be used and maintained by trained personnel. Improper use can cause injury and damage. Keep Manual with stretcher. Store Manual in separate plastic bag.

1. **Unassisted rescue -one person drag technique.**
Place victim on stretcher, put feet and head in pockets which are at both ends.

2. **Two person rescue.** When using as a two person rescue device, put feet and head in pockets which are at both ends, interlock the handles nearest the head, and interlock the second set of handles from the foot end, leaving one hand free for orientation.

The handles interlock by placing them together and pushing one flap through the other.

The middle handles can be tied together to secure the victim.

Holes are provided at both ends for drainage during water rescues.

A backboard can be inserted into the pockets.

3. **Fold the stretcher in half and interlock the handles for use as a chair carrier.**

WARNING

-Res-Q-Mate Manta is not to be used as a restraining product during vehicle transport.

-DESIGNED FOR HAND USE ONLY.

-Do not use mechanical lifting devices. Using mechanical lifting devices may cause damage to the product or injury to patient or user.

If used as a pole stretcher use Tested poles only.

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


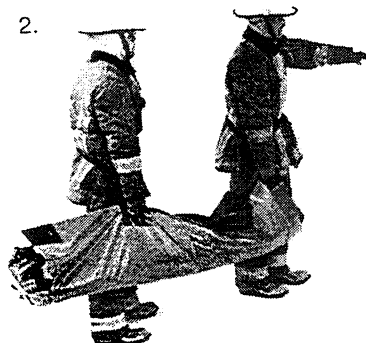
FOLDING INSTRUCTIONS BELOW

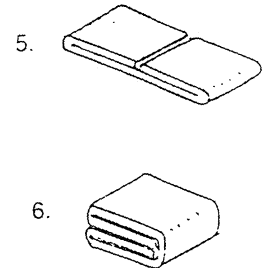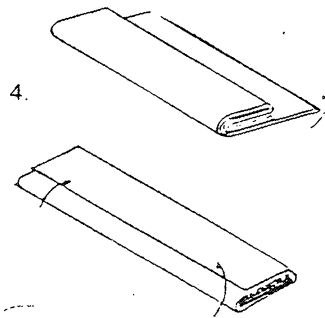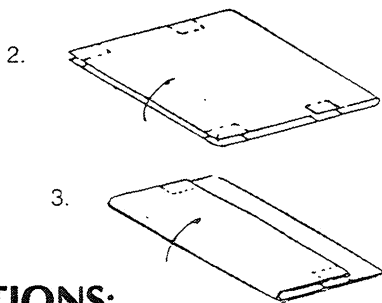1. Place the stretcher on the floor with the pockets facing up.
2. Fold the stretcher in half and overlap the handles.
3. Fold the stretcher again so that all handles overlap.
4. Fold the remaining sections over top of the handles.
5. Fold the ends of the folded stretcher so that they meet near the middle.
6. Fold the stretcher one more time.

SPECIFICATIONS:

- **Storage requirements:**
6 stretchers per cubic foot
- **Size:**
 - length, stretcher position
79 ins. (200 cm)
 - width, stretcher position
46 ins. (107 cm)
 - storage size
12 ins. X 9 ins. X 3 ins.
(30.5 cm x 23 cm x 7.5 cm)
- **Weight:**
5 lb. (2.2 kg)
- **Load Limit:**
maximum load 800 lb.(363 kg),
utilizing all 8 handles
- **Handles:** 4 per side
- **Pockets:** at both ends
- **Material:** 18 oz. Premium PVC Fabric,
with excellent UV and abrasion
resistance and industry high cold crack
of -67 degrees F.
- **Fire Resistance:**
 - material conforms to the California
State Fire Marshall's code and small
flame requirements set out by NFPA
701, ULC S-109
- **Maintenance:**
 - clean stretcher with warm water and a
mild detergent. Rinse with warm
water.
 - check for damage and excessive wear
after each use

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WARNING:

Do not repair stretcher if damaged. Replace when necessary.

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a suivre...

Limited Warranty:

J.T.W. Rescue-Stretcher Inc. warrants the product we manufacture to be free from defects in material and workmanship for 90 days.

This limited warranty applies when you use and care for the product properly. If the product is not cared for properly, the warranty is void. This warranty period begins the day the product is shipped from Ferno or the day you receive it, if you have proof of the delivery date. Shipping charges are not covered by the limited warranty. We are not liable for shipping damages or damages sustained through using the product.

Limited Warranty Obligation

If the product is found to be defective, J.T.W. Rescue-Stretcher Inc. will repair or replace it. At our option, we will refund the products purchase price. The purchaser accepts these terms in lieu of all damages.

Limitation of Liability

The liability of J.T.W. Rescue-Stretcher Inc., under the limited warranty set forth above is limited to repair or replacement of the defective product, or, at the option of J.T.W. Rescue-Stretcher Inc., refund of the purchase price.

Except for the limited warranty set forth above, J.T.W. Rescue-Stretcher Inc. makes no other warranties, either expressed or implied. J.T.W. Rescue-Stretcher Inc. makes no implied warranties of merchantability and fitness for a particular purpose. In no event will J.T.W. Rescue-Stretcher Inc. be liable for any indirect or consequential damages sustained in connection with the delivery, use or performance of its products.

The remedies of repair, replacement of defective products, or refund of the purchase price, are exclusive and in lieu of all damages. Under no circumstances shall the liability of J.T.W. Rescue-Stretcher Inc., whether under contract, tort, breach of warranty, or otherwise, exceed the selling price of the products proven to be defective.

Any legal action for breach of the limited warranty set forth above must be commenced within one year of the date on which the breach is or should have been discovered.

Limited Warranty Claims

To make a claim under the limited warranty contact J.T.W. Rescue-Stretcher Inc., R.R. #1, Lakefield, Ontario, Canada K0L 2H0. Telephone (705) 657-2854.

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