The Res-Q-Mate Manta is to be used and maintained by trained personnel. Improper use can cause injury and damage. Keep manual with stretcher. Store manual in separate plastic bag.

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Features and Benefits:

- Specially designed handles
 - handles interlock by placing handles together and pushing one flap through the other
- Pockets at both ends
 - holds victim's feet or head
 - holds back board
- Eases transfer of patient
 - victims can be transferred directly onto wheeled cot into the ambulance and on to hospital without removal from the stretcher.
- Confined space rescue
 - adapts to bathtubs, bedrooms, corners, narrow hallways, stair landings, etc.
- Material
 - fiber reinforced vinyl
 - barrier to body fluids
 - no stitching
 - fire retardant (NFPA 701/ULC-S109)
 - Manta maximum load limit 800 lb (363 kg), utilizing all 8 handles
 - Mega Manta maximum load limit 1600 lb (727 kg), utilizing all handles

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Using the Res-Q-Mate Manta:

1. Unassisted rescue – one person drag technique

Place the victim on stretcher, put feet and head in pockets which are at both ends.

2. Two-person rescue.

When using as a two-person rescue device, put feet and head in pockets which are at both ends, interlock the handles nearest the head, and interlock the second set of handles from the foot end, leaving one hand free for orientation.

The handles interlock by placing them together and pushing one flap through the other.

The middle handles can be tied together to secure the victim.

Holes are provided at both ends for drainage during water rescues.

A backboard can be inserted into the pockets.

3. Fold the stretcher in half and interlock the handles for use as a chair carrier.

WARNING

- The Manta is not to be used as a restraining product during vehicle transport
- DESIGNED FOR HAND USE ONLY
- Do not use mechanical lifting devices. Using mechanical lifting devices may cause damage to the product or injury to patient or user.
- If used as a pole stretcher use TESTED POLES ONLY

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


Folding instructions below:

1. Place the stretcher on the floor with the pockets facing up
2. Fold the stretcher in half and overlap the handles
3. Fold the stretcher again so that all handles overlap
4. Fold the remaining sections over top of the handles
5. Fold the ends of the folded stretcher so that they meet near the middle
6. Fold the stretcher one more time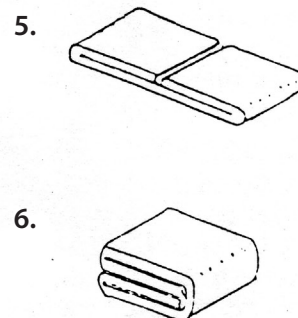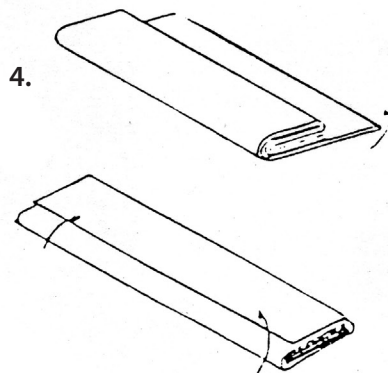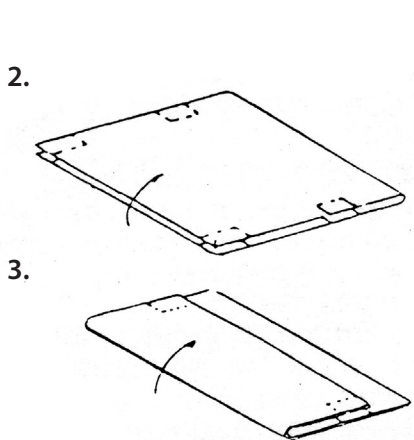

Manta Specification	Imperial	Metric
Dimensions	78 x 46 in	198 x 107 cm
Storage Size	12 x 9 x 3 in	30.5 x 23 x 7.5 cm
Storage Requirements	6 Manta/cubic ft	
Weight	5 lb	2.2 kg
Load Limit	800 lb*	363 kg*
Handles	4 per side	
Pockets	At both ends	
Material	18 oz. premium PVC fabric, UV and abrasion resistance, industry high cold crack of -67°F	
Fire Resistance	Conforms to the California State Fire Marshall's code and small flame requirements, NFPA 701, ULC S-109	
Maintenance	Clean stretcher with warm water and a mild detergent. Rinse. Check for damage and excessive wear after each use.	

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Manta and Mega Manta User Manual

Mega Manta Specification	Imperial	Metric
Dimensions	89.5 x 60 in	228 x 115 cm
Weight	8 lb	3.6 kg
Load Limit	1600 lb*	727 kg*
Handles	5 per side, 2 per end	
Pockets	At both ends	
Material	18 oz. premium PVC fabric, UV and abrasion resistance, industry high cold crack of -67°F	
Fire Resistance	Conforms to the California State Fire Marshall's code and small flame requirements, NFPA 701, ULC S-109	
Maintenance	Clean stretcher with warm water and a mild detergent. Rinse. Check for damage and excessive wear after each use.	

Note: *Utilizing all handles.

Limited Warranty:

Ferno warrants the product we manufacture to be free from defects in material and workmanship for 90 days. This limited warranty applies when you use and care for the product properly. If the product is not cared for properly, the warranty is void. This warranty period begins the day the product is shipped from Ferno or the day you receive it, if you have proof of the delivery date. Shipping charges are not covered by the limited warranty. We are not liable for shipping damages or damages sustained through using the product.

Limited Warranty Obligation:

If the product is found to be defective, Ferno will repair or replace it. At our option, we will refund the product's purchase price. The purchaser accepts these terms in lieu of all damages.

Limitation of Liability:

The liability of Ferno, under the limited warranty set forth above is limited to repair or replacement of the defective product, or, at the option of Ferno, refund of the purchase price. Except for the limited warranty set forth above, Ferno makes no other warranties, either expressed or implied. Ferno makes no implied warranties of merchantability and fitness for a particular purpose. In no event will Ferno be liable for any in direct or consequential damages sustained in connection with the delivery, use or performance of its products. The remedies of repair, replacement of defective products, or refund of the purchase price, are exclusive and in lieu of all damages. Under no circumstances shall the liability of Ferno, whether under contract, or, breach of warranty, or otherwise, exceed the selling price of the product proven to be defective. Any legal action for breach of the limited warranty set forth above must be commenced within one year of the date on which the breach is or should have been discovered.

Limited Warranty Claims:

To make a claim under the limited warranty, contact Ferno Canada.

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U.S. PATENT 5,214,813 OTHER PATENTS PENDING
Manufactured in Canada / Fabriqué au Canada

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Ferno Canada Inc.
Ferno (UK) Ltd.