USER MANUAL

Germa Kangoofix

English: User Manual **Germa Kangoofix**
Deutsch: Bedienungsanleitung, **Germa Kangoofix**
Français: Mode d'emploi, **Germa Kangoofix**
Italiano: Istruzioni per l'uso, **Germa Kangoofix**
Nederlands: Gebruiksaanwijzing. **Germa Kangoofix**
Suomi: Käyttöohje. **Germa Kangoofix**
Svenska: Bruksanvisning. **Germa Kangoofix**
Norsk: Brukerveiledning. **Germa Kangoofix**
Dansk: Brugsanvisning. **Germa Kangoofix**

Manufacturer:

AB Germa
Industrigatan 54-56
SE-291 36 Kristianstad
Sweden

Tel: +46 (0)44 12 30 30

Fax: +46 (0)44 10 31 79

E-mail: info@germa.se

www.germa.se

English:	Directions for use	Page	3
Deutsch:	Bedienungsanleitung	Seite	10
Français:	Mode d'emploi	Page	17
Italiano:	Istruzioni per l'uso	Pagina	24
Nederlands:	Gebruiksaanwijzing	Bladzijde	31
Suomi:	Käyttöohje	Sivu	38
Svenska:	Bruksanvisning	Sida	45
Norsk:	Brukerveiledning	Side	49
Dansk:	Brugsanvisning	Side	53

English: These directions for use may be updated without further notice.
Copies of the current version are available from the manufacturer.

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1. Warning and Caution statements

CAUTION

- Always ensure proper training of the Kangoofix equipment is carried out before using the product.
- Ensure that the inner harness, warming cover and skull cap (“limited use items”) which are made from a grey protective “Breath-o-prene Oeko-Tex 100 (Neoprene type)” material are cleaned prior to use, for infection control reasons
- Check on a regular basis that all “limited use items” are in good condition and check that all VELCRO® parts are undamaged and fix together properly
- NOTE the “limited use items” are **limited to 4 washes maximum** (5 uses in total), after which they should be disposed of and replaced for new ones. This for safety reasons as well as infection control
- Always check that the Kangoofix 5-point safety belts / outer harness are in good condition, have no damage and is cleaned before use. Make sure to test the buckle functionality on a regular basis. Replace outer harness immediately if there are any signs of visible damage or the buckles are not working
- Avoid handling Kangoofix system around sharp objects
- Avoid handling the Kangoofix system close to a naked flame
- Check regularly that the neck/back support plate is not damaged or showing signs of wear and tear. Replace the support plate if there are any signs of visible damage
- Kangoofix is designed to carry a neonate from approx 1.6 kg up to maximum of around 5 kg, also according to the physical body mass of the baby
- If the Kangoofix device has been involved in an accident then the unit should not be used again and should be immediately disposed of
- Recommendation: Usable life of 10 years on the restraint system
5 years on Breath-o-prene parts, Inner harness and warming bag with skull cap

Note! Breath-o-prene part to be exchanged after max 4 washes

2. Intended use

The Germa Kangoofix system is designed to comfortably and safely hold a new born baby/child during transport on an ambulance trolley/strether/cot, together with a parent in close contact. The baby is held to the parent by attaching to the trolley harness on which the parent is laid through the Kangoofix inner and outer harness system. As such it is not necessary to separate the parent from the child when transporting from one location to another, ensuring that the parent and baby are comfortable, content and safe together. If medical conditions require that the baby needs to be accessed quickly during transport, this can be achieved by using the quick release feature. The baby can also be turned and held face-up in Kangoofix, for monitoring by the paramedic during the journey, if required due to medical circumstances.

Important: In face-up direction it is important to have control over the baby's head at all times, only use this position when medical monitoring is required.

3. Specifications

The Germa Kangoofix conforms to standards ISO 6940:1984, SiS EN 1865, SiS EN 1789 and TSFS 2010:2
The Germa Kangoofix conforms to Council Directive 93/42/EEC, relating to Class 1 Medical Devices

Operating Temperature range: -35 °C to +70 °C

Parts/Materials

Inner harness, Warming cover and Skull-cap material: Breath-o-prene Oeko-Tex 100 (approved)

Belt set/outer harness: Polyester webbing

Buckles: Metal and plastic, approved for children car seat.

Dimensions	Length (cm)	Width (cm)	Height (cm)	Weight (kg)
Germa Kangoofix Inner harness	71	58	0,4	0,12
Germa Kangoofix Warming cover + Skull-cap	43	36	0,4	0,19
Germa Kangoofix neck/back support plate	25	13	0,4	0,12
Germa Kangoofix outer harness	110	93	2,4	0,58

4. Deployment of Kangoofix

The Germa Kangoofix consists of an inner harness, warming cover and Skull-cap, and a 5-point "outer-harness" safety restraint. The inner harness, warming cover and Skull cap are all made of a material called Breath-o-pren Oeko-Tex 100, which is a multi-layer grey and red breathable fabric, clinically tested and proven to cause no reaction to the baby's skin. A special hook fastening system sewn into the material allows fixing of the inner harness and warming cover around the child. The 5-point fixation restraint system is made from grey polyester webbing, with approved crash-tested metal and plastic child seat buckles.

When deploying Kangoofix, first ensure that the outer harness is attached to the trolley harness.

With the parent laid on the trolley with the backrest at a comfortable angle and the parent correctly and safely restrained using the trolley 4-point harness and leg restraints, start to attach the Kangoofix outer-harness.

The 5 grey Kangoofix outer-harness straps should be attached to the trolley harness by feeding each individual Kangoofix strap around the trolley harness strap and then back through the loop at the end of the Kangoofix strap, and then pulled tight. A Kangoofix strap should be attached to each of the trolley 4-point harness straps; around the shoulder straps each side of the patient and onto the waist straps each side of the patient's waist, with the fifth Kangoofix strap attached to the trolley upper-leg strap.

The Kangoofix inner harness and warming cover should then be wrapped around the baby, firstly ensuring that the head/back support is correctly located inside the holding pouch of the inner-harness.

Wrap the inner harness around the baby, with the red side of the inner-harness against the baby's skin, the grey part facing outwards, using the circular hook fixation pads to attach the inner harness together at the front. Start by fixing the inner harness around the baby's legs, then around the shoulders. The inner harness should be deployed and adjusted such that it is firmly held around the baby with no slack, but not too tight.

Make sure that the baby is correctly support by the neck/back support plate in the inner harness.

Next place the baby inside the warming cover, with the orientation such that the hook fixing pads on the outside of the warming cover are facing downwards, ensuring that the flaps at the front of the warming cover are then wrapped around the baby and fixed together by use of the hook fixation pads on the inner harness, to form a cocoon around the baby, with orientation such that the red side of the material is against the baby's skin.

Then place the skull cap onto the baby's head.

Note that using the warming cover can be omitted from use, if not required, and just use the inner harness on its own.

4. Deployment of Kangoofix (cont...)

Next step is to place the baby with the parent laid on the trolley, baby's face towards parent and positioned around the upper chest of the parent, away from any trolley restraint buckles or straps.

The Kangoofix quick-release central buckle should then be fixed onto the hook fixation pad on the back of the warming cover (or to the back of the inner harness if a warming mattress is not being used).

Ensure that the parent and baby are correctly and comfortably positioned, then start to connect the Kangoofix outer harness straps to the central buckle located on the back of the warming mattress (inner harness).

One by one, attach the 5 Kangoofix straps to the central buckle, starting with the straps that connect to the trolley waist straps, then from the leg strap, then shoulder straps. As each strap is attached, tighten it sufficiently such that there is no slack in the strap, still making sure that the baby and parent are comfortable and in correct position.

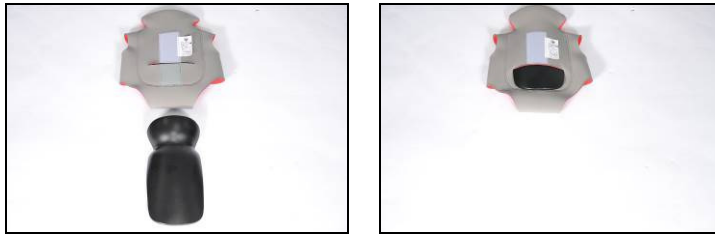
You may need to re-adjust and further tighten the straps to ensure the baby is correctly positioned and each Kangoofix strap adjusted such that the baby "cocoon" does not move around.

Refer to the "Kangoofix quick-guide" for photos and more details on attaching the Kangoofix straps and inner harness.


5. Step-by-Step Operating instructions

Below is a simplified step-by step procedure for deploying Kangoofix

1. Importantly first make sure you have the neck/back protection plate is inside the Inner Harness before use

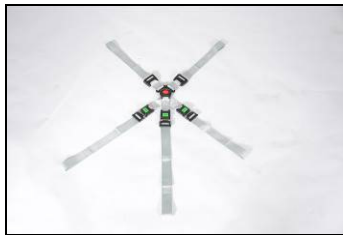2. Disconnect each of the 5 safety belt straps from the 5-point quick-release buckle.

3. Connect Kangoofix outer-harness shoulder straps, waist and upper leg belt straps by making loops around the trolley 4-point harnesses and upper leg harness.

4. Open up the inner harness, place the baby with his/her back towards the red part of inner-harness, making sure that the baby is correctly supported by the neck/back support plate in the inner harness.

5. Start to close the inner harness around the child, starting with the flap between the legs and fix this part to one of the shoulder flaps, using the attached hook fixing pads

5. Step-by-Step Operating instructions (cont...)

6. Close the side flaps of the inner harness to the centre, again using the hook fixing pads, make sure the harness is tight but baby comfortable.

NOTE: Connect shoulder flaps before the side flaps

7. Make sure that the child is comfortable in the inner harness and then wrap the warming cover around the baby, and place the skull cap on, making sure to cover the child safely and comfortably. Note that the warming cover can also be wrapped around the baby after he/she has been placed on top of the parent on the trolley.

8. Place baby with mother on the trolley, face down and connect the Germa Kangoofix central quick release buckle to the back of the warming cover.

9. Connect the 2 Kangoofix outer-harness waist straps to the central 5-point buckle and tighten to remove the slack in the straps
10. Connect the Kangoofix leg strap to the central 5-point buckle and tighten to remove the slack
11. Connect the Kangoofix shoulder straps to the central 5-point buckle and tighten to remove the slack, making sure the baby is comfortably positioned

12. Make sure baby and parent are in the required and comfortable position, and start to gently further tighten the 5 straps such that the baby is firmly positioned

NOTE! In a medical emergency where immediate intervention to the baby is required, press the RED quick release button on the central buckle to free the baby.

6. Cleaning and disinfecting

After each use all parts of the Kangoofix system should be machine washed in a washing bag, using good quality standard washing powder. They items should be washed at 60°C. Do not iron and no dry cleaning.

Allow the items to dry completely before storing and next use. Do not tumble dry.

NOTE - There is a maximum of 4 washes for the Inner harness, warming cover and skull cap (5 uses max) after which the items should be replaced, for safety reasons.

NOTE: - Labels on the Inner Harness and on the warming cover with attached skull cap allows a record to be made each time an item is washed, and allows it to be tracked as to when an item needs replacing (after 4 washes/5 uses)

The outer harness can be washed and re-used an unlimited number of times as long as they continue to show no signs of wear and tear.

If the Kangoofix requires disinfecting, first clean it carefully and let it dry. Wipe the item with a solution containing 70% alcohol or use the disinfectant called LYSETOL AF.

Test all Kangoofix parts at regular intervals to make sure that they are working well, are not damaged in any way and do not show any signs of wear and tear. If any part does show signs of wear or damage, then it should not be used anymore and replaced immediately

7. Maintenance and repair

At a regular intervals and prior to each use Kangoofix must be checked for any visible damage, good functionality and that it is clean.

The Inner harness and Warming cover with Skull cap should be checked that they are in good condition after washing and the hook fixing pads tested to ensure they work properly and make a good locking/fixation.

At the same time the functioning of 5-point belt set should be tested, by opening and closing of all buckles to make sure they work safely. Furthermore the quick release buckle and button should be checked for correct function/

If any of the component parts of Kangoofix is showing any kind of diminished function or damage then it should be replaced immediately.

NOTE: Never repair any parts of the Kangoofix system for safety reasons.

8. Testing of Kangoofix

To test Kangoofix, open and close of all buckles to make sure they work properly.

All outer harness straps must be tested and checked that they are working correctly

Also ensure that the quick release buckle and button are functioning correctly.

Inner harness. Warming cover and Skull cap hook fixing pads should be tested by visually inspection and function test of to ensure they close and fixate properly, that there is no damage to the hook fixing pad surface and that they are not dirty.

9. Storage

Make sure that once all component parts of the Kangoofix system have been checked that they are all stored together in a suitable bag,

The bag should include:

- Complete 5-point belt set system with central 5-point buckle and 5 fixing straps
- Inner harness with neck/back support plate inserted inside
- Warming cover with Skull cap attached

Ensure to store the Kangoofix in a dry and clean place.

10. Ordering information – Kangoofix

15 271 000 819 Kangoofix Paediatric restraint system kit (including storage bag)

15 271 000 820 Kangoofix Paediatric restraint system kit

15 271 000 826 Kangoofix "limited-use" items replacement kit (inner harness, warming cover & skull-cap)

11. Spare parts

15 271 000 821 Kangoofix replacement outer harness

15 271 000 822 Kangoofix replacement inner harness

15 271 000 823 Kangoofix replacement warming cover with skull cap

15 271 000 824 Kangoofix replacement neck/back support plate

15 271 000 826 Kangoofix replacement inner harness + Kangoofix replacement warming cover with skull cap

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