AB GERMA®

INSTRUCTIONS FOR USE

KANGOOFIX

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KANGOOFIX

Introduction

Kangoofix is a safety belt system for safe ambulance transport of a newborn baby/child placed on top of an adult. Medical studies prove that patient management of babies is improved when skin contact with the parent can be achieved.

The adult shall be placed on a stretcher/stretcher making use of its 4-point safety belt system and leg safety belt where Kangoofix is fixated. For any unexpected situation with the child or parent the harness system has a quick release button to free the baby.

Expected life service: 10 years of the restraint system.
5 years of the Breath-o-prene parts or after 5 uses (and 4 washes).

Precautions

- Always ensure proper training of the Kangoofix system is carried out before using the product.
- Ensure that the inner harness, warming cover and skull cap (limited-use items), which are made from protective 'Breath-o-prene Oeko-Tex 100 (Neoprene-type) material, are cleaned prior to use for infection control purposes.
- Check regularly that all limited use items are in good condition and ensure that all hook-and-loop fasteners are undamaged and fasten together properly.
- The limited use items are limited to 4 washes maximum (5 uses in total), after which they should be disposed of and replaced for new ones. This is for safety reasons as well as infection control.
- Always check that the Kangoofix 5-point safety belt (outer harness) is in good condition, free from damage, and cleaned before use. Make sure to test the buckle functionality regularly. Replace the outer harness immediately if there are any signs of visible damage or if the buckles are not functioning properly.
- Always use the provided Triglide belt adjusters if the stretcher's safety belt system does not include shoulder belts with built-in belt adjusters.
- Check the neck/back support plate regularly to ensure it is not damaged or showing signs of wear and tear. Replace the support plate immediately if any visible damage is detected.
- If the Kangoofix device has been involved in an accident, it should not be used again and must be disposed of immediately.
- Avoid handling the Kangoofix system around sharp objects.
- Avoid handling the Kangoofix system near an open flame.
- Own modifications on products are not allowed – Your medical specialist is to be responsible for any modification.
- Do not tumble dry or use a heated drying cabinet.

Warnings

When using the face-up position, it is important to maintain control of the baby's head at all times. This position should only be used when medical monitoring is required.

The Kangoofix is intended for neonates weighing from 1.6 kg up to 5 kg, with the exact suitability also depending on the infant's physique.

Description

The Kangoofix system consists of:

- an inner harness,
- a warming cover with a skull cap
- a 5-point safety restraint (outer harness).

The inner harness, warming cover with skull cap are all made of a material called Breath-o-prene Oeko-Tex 100, a multi-layer black breathable fabric that has been clinically tested and proven not to cause any reaction to the baby's skin.

A special hook fastening system sewn into the material allows the inner harness and warming cover to be securely fixed around the child. The outer harness is made from grey polyester webbing and features approved, crash-tested metal and plastic child seat buckles.

Stretcher shoulder safety belts without belt adjusters

If the stretcher's shoulder safety belts lack belt adjusters, the provided Triglade belt adjusters shall first be assembled before using Kangoofix system. Assemble the Triglade belt adjusters at upper chest height on the left and right shoulder safety belt.

Note that in following pictures, the Triglade belt adjuster is mounted too close to the buckle (the buckle is used as a guidance to clarify the assembly procedure).

1. Enter belt through the lower slit of the belt adjuster, make sure the arched side of belt adjuster is directed from the male buckle:

2. Enter belt through the upper slit of the belt adjuster:

3. Loop the belt and turn back on arched side of belt adjuster, and through the lower slit of the belt adjuster:

Healthcare professionals must adjust the positioning according to the backrest angle and the size of the parent. They must position the belt adjusters so that the infant's head does not come into contact with the belts, and so that the infant's head can be monitored at all times.

Once the Triglade belt adjusters are in place, on both shoulder belts, the Kangoofix outer harness straps can be connected to the stretcher's belt system, see next section.

Mounting the Kangoofix Outer Harness to the stretcher belt

1. Ensure the outer harness (5-point restraint) is in good condition and clean.
2. Place the adult patient on the stretcher and secure them with the stretcher's 4-point and leg safety belts.
3. Attach the five grey Kangoofix outer-harness straps to the stretcher harness by feeding each strap around the corresponding stretcher belt, above the stretcher belt adjuster, back through the loop at the strap end, and pulling it tight (see illustration next side):

- Two straps to the shoulder belts
- Two straps to the waist belts
- One strap to the upper-leg belt

4. Make sure all straps are flat, untwisted, and securely fastened.

5. Adjust the stretcher's backrest and belt adjusters as needed for parent size and comfort.

Securing the baby using the KangooFix harness system

1. Ensure that the neck/back support plate is correctly positioned inside the inner harness before use.

2. Place the baby on the open inner harness with the hook-and-loop fasteners facing outward, then close the flap between the legs (marked no. 1);

INSTRUCTIONS FOR USE

KANGOOFIX

3. Close the shoulder flaps (marked no. 2 & 3);

4. Close the side flaps (no. 4 & 5):

Adjust for a firm but comfortable fit.

5. Wrap the warming cover around the baby and place the skull cap on the baby's head.
(The warming cover can also be wrapped around the baby after the baby has been placed on top of the parent on the stretcher.)

The warming cover may be omitted if not required.

6. Place the baby on the parent's chest, facing down (skin-to-skin), or face-up if medical monitoring requires it.

Ensure the baby's head is always visible and supported!

7. Attach the central quick-release buckle of the outer harness to the hook fixation pad on the back of the warming cover (or inner harness).

8. Connect the five outer harness straps to the central buckle in the following order:

- Waist straps
- Leg strap
- Shoulder straps

Tighten each strap evenly to remove slack while ensuring comfort.

9. Verify that both parent and baby are correctly positioned and stable before transport.

*In a medical emergency requiring immediate intervention, press the **red quick-release button** on the central buckle to release the baby.*

Cleaning and disinfection

After each use, all parts of the Kangoofix system shall be machine washed in a washing bag using a good-quality, standard washing powder. The items should be washed at 60 °C.

Do not iron or dry clean.

Allow the items to air dry completely before storing and before the next use.

Do not tumble dry or use a heated drying cabinet.

The inner harness, warming cover, and skull cap *can be washed a maximum of four times* (up to five uses). After this, the items shall be replaced for safety reasons. If any part shows signs of wear and tear earlier, it should be replaced immediately.

INSTRUCTIONS FOR USE

KANGOOFIX

Labels on the inner harness and on the warming cover with attached skull cap allow users to record each wash and help track when the item needs to be replaced (after a maximum of four washes or five uses).

The outer harness can be washed and reused an unlimited number of times, as long as it continues to show no signs of wear and tear.

Test all Kangoofix parts after each use to ensure that they are functioning correctly, are not damaged, and show no signs of wear and tear. If any part does show signs of wear or damage, it should no longer be used and must be replaced immediately.

INSTRUCTIONS FOR USE

KANGOOFIX

Maintenance and repair

After each use (to ensure it is ready for the next use), the Kangoofix must be checked for any visible damage, proper functionality, and cleanliness.

The inner harness and warming cover with skull cap should be inspected to ensure they are in good condition after washing. The hook-and-loop fixation pads should also be tested to ensure they function correctly and provide secure locking.

At the same time, the functionality of the outer harness should be tested by opening and closing all buckles to confirm they operate safely. Additionally, the quick-release buckle and button should be checked to ensure correct operation.

If any component of the Kangoofix shows signs of diminished function or damage, it must be replaced immediately.

Never attempt to repair any part of the Kangoofix system for safety reasons.

When deciding to take the product out of service and destroy it, make sure to follow your local regulations or contact us for further information.

Storage

After all components of the Kangoofix system have been checked, ensure that they are stored together in its storage bag. Store the Kangoofix in a clean, dry place.

The bag shall include:

Complete outer harness

Inner harness with neck/back support plate inserted

Warming cover with skull cap attached

The Germa Kangoofix complies with the requirements of EN 1789.

The product has been tested and meets the requirements of TSFS 2016:22, Annex 3, Section 4 (P1).

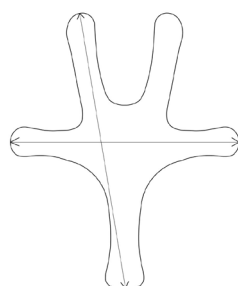
The Germa Kangoofix is compliant with the EU Medical Device Regulation (MDR 2017/745) as a Class I medical device.

Parts/Materials

- Inner harness, warming cover, and skull cap: Breath-o-prene Oeko-Tex 100 approved
- Belt set (outer harness): Polyester webbing
- Buckles: metal and plastic, approved for use in child car seats

Dimensions

Dimension of the inner harness is 71 (length) x 58 (width) cm:

Products

Art. no.	UDI-DI	Product name
15271000819	07350019596081	Safety Belt System Kangoofix Pediatric Ready2go*
15271000820	07350019596098	Safety Belt System Kangoofix Pediatric Ready2go**

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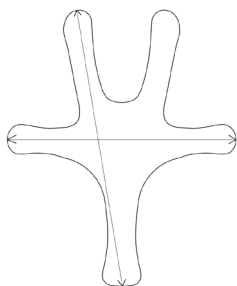
NOTICE TO USERS AND PATIENTS

In the event of a serious incident related to the use of this device, it is imperative to take the following steps:

1. Report the Incident to the Manufacturer:

Contact the manufacturer of the device promptly. Ensure that all relevant details about the incident, including the circumstances and any observed effects, are provided to facilitate investigation and resolution.

2. Notify the Competent Authority:

The competent authority in the countries where you are established must also be informed about the incident. This ensures compliance with regulations and enables authorities to take any necessary actions to safeguard public health.

Your prompt reporting helps improve device safety and ensures appropriate measures are taken to prevent future occurrences. Thank you for your cooperation.