
Model 418-1 Lifting Bridle

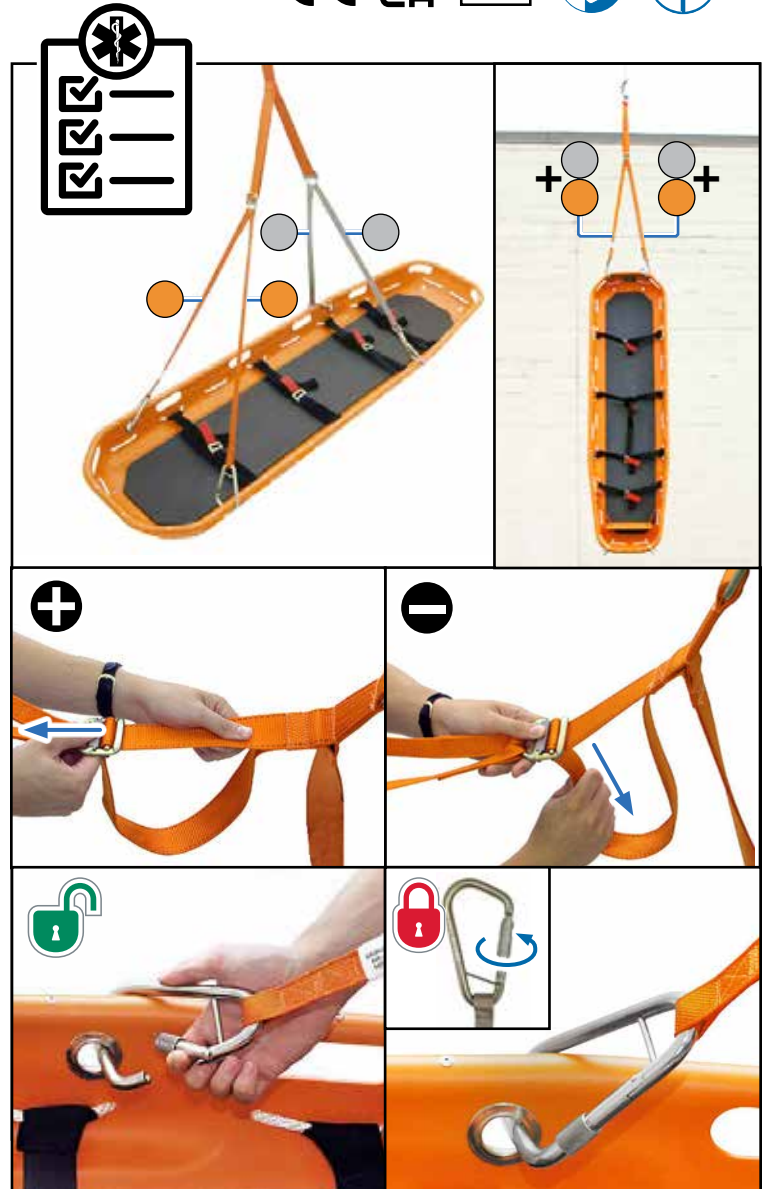
Product	
A	54.5" (1384 mm)
B	34.5" (876 mm)
C	2.625" (67 mm)
D	3" (76 mm)

	5 lb (2 kg)
--	-------------

	600 lb (272 kg)
--	-----------------

	-40°F-140°F (-40°C-60°C)
--	--------------------------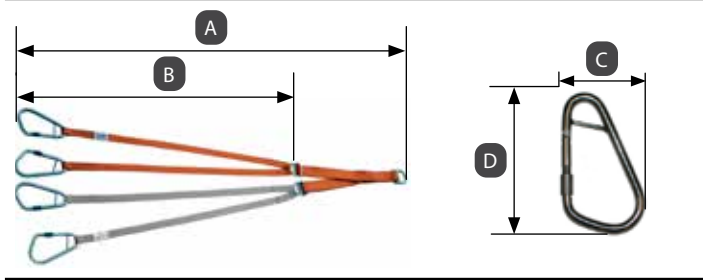

Ferno-Washington, Inc.
70 Weil Way
Wilmington, OH 45177-9371, U.S.A.
1.937.382.1451

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Description

Model 418-1 is an adjustable lifting bridle designed to be used as an interface between a basket stretcher and a rope hoist, cable, or other lifting device. It is compatible with Ferno models 71 and 71-S basket stretchers.

Maximum range of adjustment for each leg is 11" (279 mm). Maximum separation of the legs (gray and orange) is 100" (254 cm).

General Guidelines for Use

Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices during use of the product. A minimum of two trained operators is required. Operators must:

- Read and understand this manual.
- Train and practice on proper use of the product following standard emergency patient-handling procedures. Keep training records.
- Rescue procedures using the lifting bridle require special skills and are potentially hazardous activities. Do not attempt rescue procedures unless operators have received professional instruction and training.
- Inspect the product and confirm it is in good condition.
- Communicate with one another and use coordinated movements to maintain control of the product. Stay with the patient at all times.
- Make sure all equipment used with the bridle has a load rating at least equivalent to that of the bridle.
- Secure patient in the stretcher using approved restraints (not included).
- For horizontal lifting, attach gray legs at one end of the stretcher and orange legs at the opposite end.
- For vertical lifting, attach one gray and one orange leg at each side.
- Secure lifting bridle to approved attachment points on the stretcher by fully tightening locking sleeve to carabiner nose.
- Store bridle in a dry place away from direct sunlight.

Disinfecting and Cleaning

Remove the bridle from the stretcher before cleaning and disinfecting.

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

Hand clean with warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry. When completely dry, lubricate carabiners with WRL-191-S synthetic lubricant or a multi-purpose grease.

Ferno Customer Relations

For ordering assistance or general information, contact your Ferno distributor or Ferno Customer Relations.

Information www.ferno.com
Authorized Service and Support ferno.com/service-and-support

U.S., Canada	1.877.733.0911
International	+1.937.382.1451
Fax	1.888.388.1349

Complete English Users' Manual
available at: www.ferno.com.

PATENTS
ferno.com/patents

WARNING

Professional use by trained operators only. Use the product only for its intended purpose.

Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.

An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.

Improper modifications, or items not designed for the product can cause injury. Attach only approved items. Do not modify the product.

Symbol	Definition	Symbol	Definition
	Follow training protocol		Load capacity
	Disinfecting		Product weight
	Cleaning		Temperature range

NOTICE

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not use abrasive materials to clean the product.

Wash by hand. Do not wash bridle by washing-machine action.

Inspection Checklist

- Are all components present, in good condition, and free of excessive wear?
- Is webbing in good condition with no cuts or frayed edges?
- Are V-shaped rings, carabiners, and lifting ring true to their original shape and free of cracks and gouges?
- Are carabiners free of visible damage, with hinge pins securely in place, and do they operate properly?

If inspection shows damage or excessive wear, remove from service and replace.

Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911 or via email at tscoordinator@ferno.com. If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

Disposal

Prior to disposal, clean and disinfect the product according to instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste.

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