

FLOTATION COLLAR, 2 PIECE [11-0124A]

ATTACHING THE FLOTATION COLLAR, 2 PIECE TO A RESCUE LITTER

1. Place collar on a flat surface. Separate left and right collars, rear straps remaining attached.
2. Place rescue litter between the left and right collars; zipper at the head end of the rescue litter with the logo visible on the patient's right.
3. Clip five short restraint straps located along right collar to the rescue litter. Repeat process with left collar. Tighten restraint straps, as required. (If installing on a Split-Apart rescue litter, ensure that the 3rd and 4th straps are on either side of the LocSafe)
4. Turn rescue litter on its front and adjust the rear straps, as required.

STORING THE FLOTATION COLLAR, 2 PIECE

1. Unclip rear straps on the underside of the rescue litter.
2. Turn the rescue litter back over and flip collars into it.